

AceptiCide RTU™

AceptiCide RTU™ is an EPA registered, one-step disinfectant/cleaner designed to remove surface contaminants, pathogens and disinfect hard, non-porous surfaces. This all in one, broad-spectrum formulation is fast-acting and effective as a detergent, cleaner, disinfectant, sporicide/sterilant, deodorizer and fungicide and has been proven to kill all forms of microbial life.

PRODUCT INFO

Active Ingredients: Hydrogen Peroxide, Peracetic Acid

Formula: Ready To Use (RTU)

Size: 32 fl. oz. X 6 per case

ITEM NO: 877587HG32 (sterile)

877587HG32NS (non-sterile)

EPA REG NO: 58300-25-96853

EPA EST. NO: 58300-FL-1

SPECIFICATIONS:

Total Actives 6.74%

Biodegradable: YES

pH value 1.5 - 2.5

Shelf Life: 2 years

Form: Liquid

Color: Clear

Odor: Mild (Fresh Mint Scent)

ORGANISM	CONTACT TIME
<i>CLOSTRIDIODES DIFFICILE (C.DIFF)</i>	3 MINUTES
<i>HUMAN CORONAVIRUS 229E</i>	2 MINUTES
<i>INFLUENZA A (H1N1)</i>	2 MINUTES
<i>INFLUENZA A (H3N2)</i>	2 MINUTES
<i>PSEUDOMONAS AERUGINOSA</i>	2 MINUTES
<i>ESCHERICHIA COLI (E.COLI)</i>	30 SECONDS
<i>SALMONELLA ENTERICA</i>	90 SECONDS
<i>STAPHYLOCOCCUS AUREUS</i>	30 SECONDS
<i>METHICILLIN RESISTANT S. AUREUS (MRSA)</i>	2 MINUTES
<i>RHINOVIRUS</i>	2 MINUTES
<i>BACILLUS SUBTILLIS</i>	20 MINUTES
<i>CLOSTRIDIUM SPOROGENES</i>	20 MINUTES
<i>NOROVIRUS</i>	2 MINUTES
<i>KLEBSIELLA AEROGENES</i>	30 SECONDS
<i>TRICHOPHYTON METAGROPHYTES</i>	1 MINUTES
<i>ASPERGILLUS NIGER</i>	2 MINUTES
<i>AVIAN INFUENZA VIRUS (H9N2)</i>	2 MINUTES
<i>CANINE PARVOVIRUS</i>	2 MINUTES

Destroys or eliminates all forms of microbial life ♦, including bacterial spores, in the inanimate environment

**This product is listed on the EPA List N: Disinfectants for use against SARS-CoV-2 (COVID-19) when used as directed.

AceptiCide RTU™ is effective against a wide range of bacteria, viruses and fungi including:	
DISINFECTANT	BACTERICIDAL (2 Minutes)
	TUBERCULOCIDAL (5 Minutes)
	Virucidal (2 Minutes)
	Sporicidal [Activity Of Disinfectant] (3 Minutes)
	Fungicidal/Mold Killing (1 Minutes) Hard Surfaces
SANITIZER	30 Seconds On Hard Inanimate Non-Food Contact Surfaces
STERILANT	25 Minutes @ Room Temperature

McGowan Industries, Inc.

DBA: Total-Shield Plus

San Diego, CA

877-910-4208

www.ACEPTICIDE.com